

BRYANRADIOLOGYASSOCIATES

2722 Osler Boulevard Bryan, TX 77802 Phone: (979) 776-8291 Fax: (979) 774-7871

3-Phase Abdomen

Indications : Liver Mass,

Scan Type	Helical
Scan Start\ End Location	Above diaphragm to iliac crest
Scan Details	5mm X 5mm abdomen window Non-contrast Arterial phase Venous phase
IV Contrast	70ml injected at 2cc/sec
Scan Delay	Start scan when descending aorta is filled with contrast or 180 HU for sure start
Reformat Details	Coronals on the venous phase
Comments	
Images to send to PACS	Scouts, all 3 phases of axials, and coronals